


Low profile two step system
that maintains consistent compression
effective in treating venous leg ulcers
and related conditions.

Visit www.andoverhealthcare.com to watch the application video.

ORDERING INFORMATION

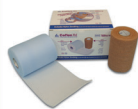

CoFlex TLC - For patients with $\geq$ ABI 0.8

Catalog #	Color(s)	Rolls/Box	Boxes/Case	Roll Size
7800TLC-TN	Tan	2	8	Layer 1: 10 cm x 3,1 m (unstretched) Layer 2: 10 cm x 4,7 m (Stretched)

CoFlex TLC Lite - For Patients with ABI $\geq$ 0.5

Catalog #	Color(s)	Rolls/Box	Boxes/Case	Roll Size
7802TLC-TN	Tan	2	8	Layer 1: 10 cm x 3,1 m (unstretched) Layer 2: 10 cm x 4,7 m (Stretched)

Distributed By:

Sourcemed

surgicals | wound care

+27 (0)21 461 2985

www.sourcemed.co.za



Andover Healthcare, Inc.
9 Fanaras Drive, Salisbury, MA 01952
www.andoverhealthcare.com
1-800-432-6686 - 978-465-0044

U.S. PATENT NO. 5.762.623
U.S. PATENT NO. 6.156.424
EUROPEAN PATENT NO. 0839222
PATENT PENDING – PUBLICATION NO.: 2012/0238933

MADE IN USA

Now Available in
COLORS!
See back for details.



CoFlex[®]·TLC
ANDOVER TWO LAYER COMPRESSION

**NEW ABSORPTIVE
FOAM LAYER**
(Patent Pending)



Easy
Hand-Tear

Continuous
restorative compression
for treating venous leg ulcers.

Nylon stocking included in each kit.



Apply in Two Easy Steps



Layer 1: Absorptive Soft Foam Layer

Apply Layer 1 with blue side against the patient's skin starting at the base of the toes, winding behind the heel leaving the plantar surface of the heel exposed and continuing up the leg with a 50% overlap.



Layer 2: Hand-Tear Cohesive Compression Layer

Begin winding at the base of the toes covering Layer 1. Cover the entire heel by wrapping 2-3 figure of eights around the ankle. Continue up the leg overlapping at 50% while maintaining consistent tension on the wrap.

Please refer to the booklet inside each box for full set of instructions.

Nylon Stocking Included!

NEW!




Apply nylon stocking over the completed dressing for patient comfort and ease of movement under clothes and on bed sheets.

CoFlex TLC Two Layer Compression

A two step compression system designed to treat venous leg ulcers and other related conditions.

Layer 1

A Soft foam with an absorbent coating that wicks away light serous drainage. Provides a rapid moisture vapor transfer rate that helps keep the peri-wound skin dry.

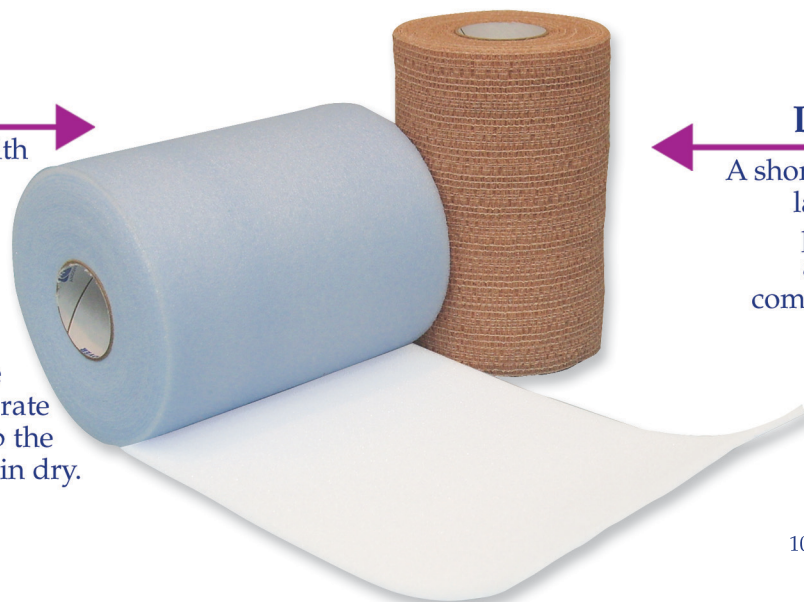

Layer 2

A short stretch layer that provides adaptive compression levels.

10 cm x 3,1 m


10 cm x 3,5 m

10 cm x 4,7 m

Both layers are latex free and bond firmly together when applied which keeps the system in place.

Please refer to the back of this brochure for ordering information.

Layer 1

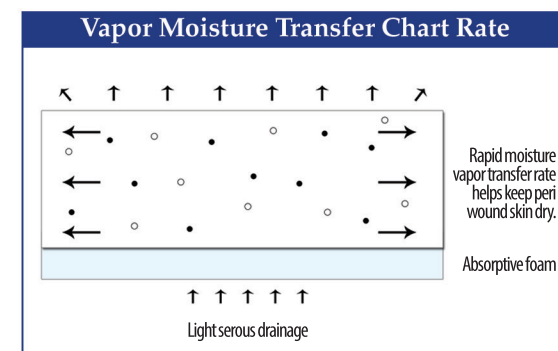

Andover's CoFlex TLC Two Layer Compression delivers adaptive compression for patients with edema associated with venous leg ulcers and related conditions. The system includes two latex free bandages that provide compression for all patient shapes and sizes. Available in colors, a Lite version for patients who are less tolerant of compression and an XL version for patients with highly contoured legs. No matter what you choose, CoFlex TLC Two Layer Compression can be worn for up to 7 days maintaining consistent compression.