



# Wound Care Solutions

*introducing!*  
... a 2-step level of  
comfort and healing unlike  
any other Unna boot with Zinc!



## CoFlex™ • UBZ Kit

NEW Unna boot kit with ZINC + Cohesive Bandage

PATENT PENDING

50% MORE  
active ingredients per roll than  
the current leading brand!

Healthcare  
Professionals  
say ...

- ✓ Finally - a new Unna boot!
- ✓ Easy & quick to apply
- ✓ Comfortable & soothing
- ✓ Stays moist - not messy
- ✓ Reduces edema & absorbs exudate
- ✓ Doctors & Clinicians love it!




**Step 1**  
Unna boot  
Foam Comfort  
Roll with Zinc  
*Not made with  
natural rubber latex*

*Absorbs  
Moisture*



**Step 2**  
Cohesive Bandage  
Moisture Resistant  
Short Stretch

*Provides  
Continuous  
Restorative Compression*

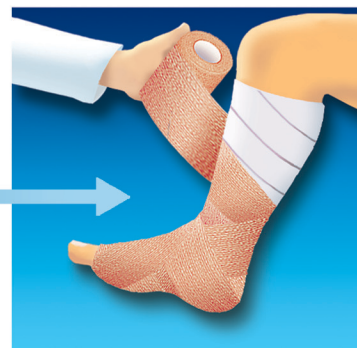

Kit includes nylon stocking

# Applications and Uses


- ✓ Venous Leg Ulcers
- ✓ Pressure Ulcers
- ✓ Diabetic Foot Ulcers
- ✓ Edema
- ✓ Lower Extremity Wounds
- ✓ Chronic Wounds
- ✓ Traumatic Wounds
- ✓ Burns
- ✓ Rashes
- ✓ Sprains
- ✓ Strains
- ✓ Skin Tears

# CoFlex™ • UBZ Kit

## Unna boot with ZINC + Cohesive Bandage

### Help for patients with Chronic Venous Insufficiency!

Chronic Venous Insufficiency (CVI) is caused when valves in the veins are weakened, which decreases the ability of the veins to pump blood up to the heart to be re-oxygenated. Compression therapy has been medically recognized as one of the most effective non-invasive therapies to treat CVI, venous ulcers, edema, and other skin irritations.

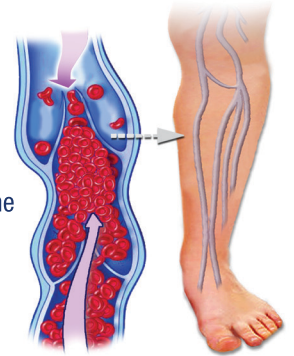

**CoFlex™•UBZ** is an innovative **2-Step** bandaging system. Quick and easy, it treats patients with a new level of comfort and healing unlike any other Unna boot. The **Step 1** foam comfort roll with zinc eases pain and skin irritation. The foam layer will absorb 200 grams of exudate per square meter, to help patients heal comfortably. The **Step 2** Cohesive Bandage delivers continuous restorative compression to assist with edema and the healing of wounds.

**CoFlex™•UBZ** delivers 20-30mmHg of pressure.

## Venous Leg Ulcer Application



### Step 1

#### CoFlex™ Unna boot Foam Comfort Roll with Zinc

- 1 Position patient's foot at a 90° angle
- 2 Begin wrapping Unna boot at the base of the toes in a spiral wrap around the foot
- 3 Wrap around ankle with 1-2 figure eights
- 4 Continue spiral wrapping - overlapping 50% up the leg ending just below the knee
- 5 Smooth down material as you wrap
- 6 Cut off excess material and discard

### Step 2

#### CoFlex™ Cohesive Bandage - provides compression

- 1 Apply Step 2 bandage to completely cover Step 1 Unna boot
- 2 Position the foot at a 90° angle
- 3 Begin wrapping bandage at the base of the toes in a spiral wrap to the heel
- 4 Wrap around the ankle covering completely with 2-3 figure eights
- 5 Continue spiral wrapping overlapping 50% up the leg ending just below the knee (adjust tension to patient's tolerance and/or physician's orders)
- 6 Cut off excess material and discard
- 7 Gently smooth the bandage to keep it in place

**Finish** - Slip the nylon stocking over the bandage for comfort and ease of movement under clothing and on bed sheets

**Duration** - CoFlex™•UBZ can remain on the patient's leg for up to 7 days

**Remove** with bandage scissors or by unwrapping

Caution: In the latex kits, the Step 2 bandage roll contains natural rubber latex which may cause allergic reactions

### Ordering Information

#### CoFlex™•UBZ 2-step kit

Unna boot with Zinc + Cohesive Bandage

**UBZ KIT - NOT MADE WITH NATURAL RUBBER LATEX:**

**4 in. Catalog# . . . . . 8840UBZ-TN**

Step 2 Color . . . . . Tan

Rolls/Box . . . . . 2

Boxes/Case . . . . . 8

Roll size Step 1 . . . . . 10 cm x 5,5 m (unstretched)

Roll size Step 2 . . . . . 10 cm x 6,4 m (stretched)

Kit includes nylon stocking



### Distributed By:

# Sourcemed

surgicals | wound care

+27 (0)21 461 2985

www.sourcemed.co.za



**Andover Healthcare, Inc.** • www.andoverhealthcare.com • **PROUDLY MADE IN USA**

U.S. PATENT NO. 5.762.623 | U.S. PATENT NO. 6.156.424 | EUROPEAN PATENT NO. 0839222 | PATENT PENDING

CoFlex® is a registered trademark of Andover Healthcare, Inc.